

Generic AED Checklist

Client Name:

Date of Unit Audit:

Unit Identifier:

Person Conducting Check:

Check the Status Indicator

Ensure the unit is indicating it is ready for use

Check the Expiration Date Pads

Check to ensure the pads currently installed are within their usage date. Ensure the pads are not open or have damaged packaging

Check the Battery

Check to ensure the battery currently installed is within the usage date

Battery Expiry Date

Check Supplies

Check accessories and spares for damage or expiration. Replace any accessories found to be damaged or that have exceeded their expiration date.

Check Exterior of the Unit

Check the unit for cracks or other signs of damage.

If any of the checks identify issues, immediately contact APTS on 1300 325 001 or email store@auspactrianing.com.au